Seating

Standing

Sleeping

Therapy


JENX[®]
Positioning for life

Instruction for use part 2 of 2

LY01

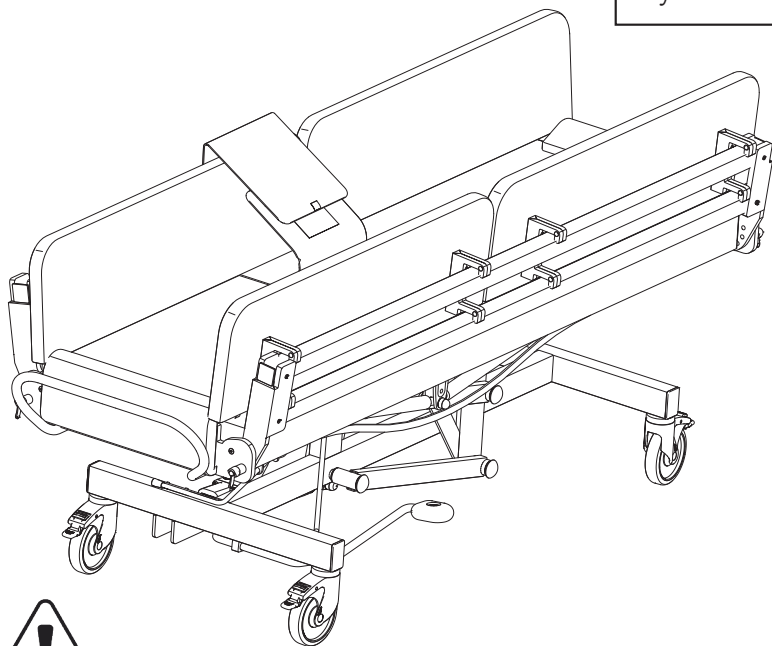May 2017. Version 003

Read part 1 and part 2 before use

Carers and home users

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- DO NOT USE as a mobility device Never use product on rough ground or uneven surface.

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WARNING

If contamination is suspected then decontaminate the couch before use.

Torn or damaged upholstery should be replaced immediately.

CAUTION

Do not use Alkalis or other organic solvents for routine cleaning.

Always cover the vinyl with paper towelling or couch covers if it can come into contact with bare skin.

Cleaning

Always wear suitable protective clothing before carrying out this procedure.

Step 1. Using warm water and soap on a damp cloth, carefully wipe the couch over from top to bottom.

Step 2. Wipe the couch carefully from top to bottom using a clean damp cloth and clean water.

Step 3. Carefully dry the couch from top to bottom with clean lint free paper towels.

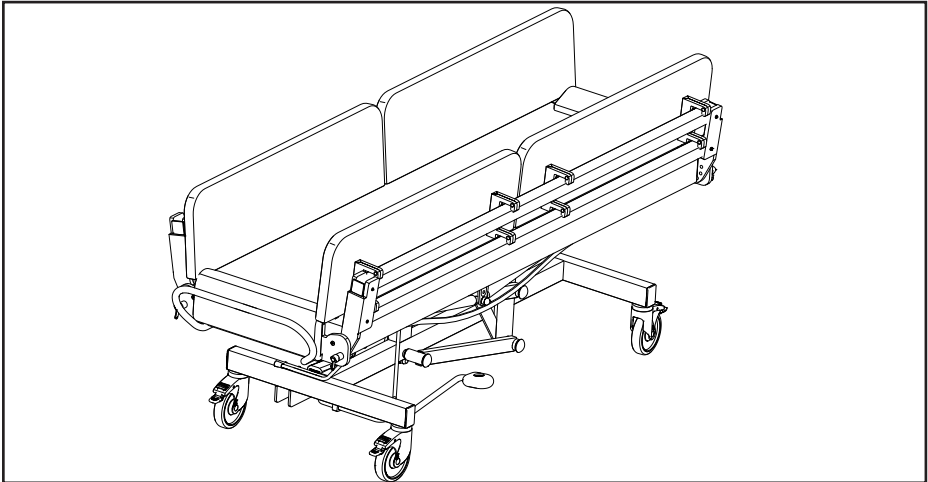
Decontamination

To decontaminate the couch repeat the above steps but using a dilute solution of Hypochlorite and warm water (10,000ppm) instead of warm water and soap. At the end of the procedure discard all cleaning materials in a safe manner and wash hands thoroughly.

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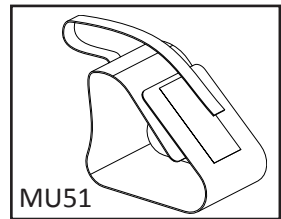
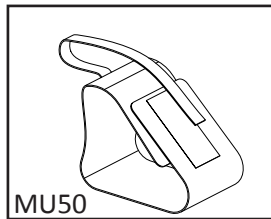
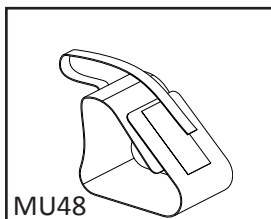
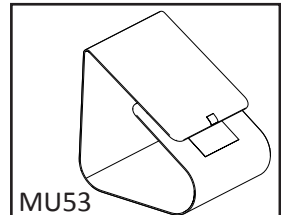
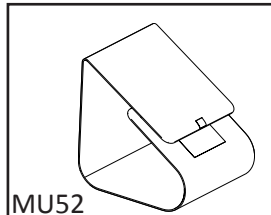
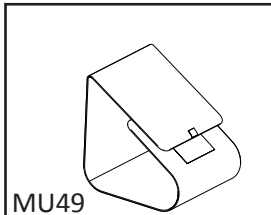
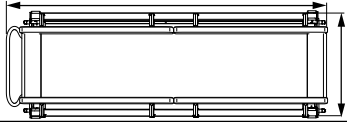
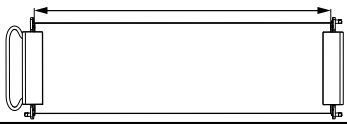
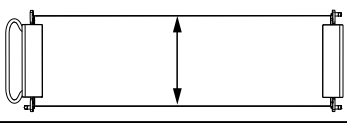
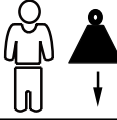
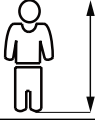
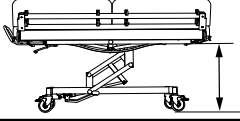
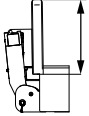
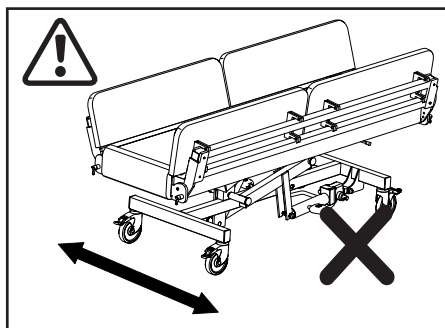
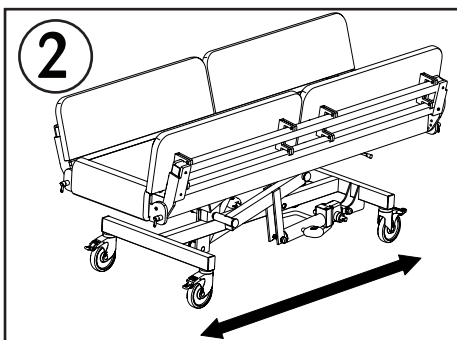
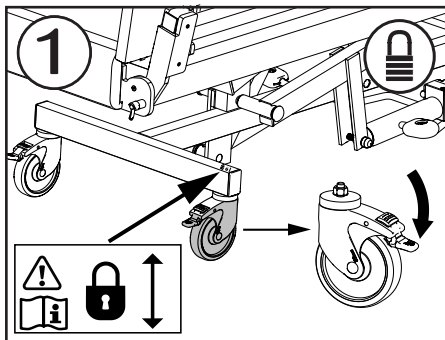
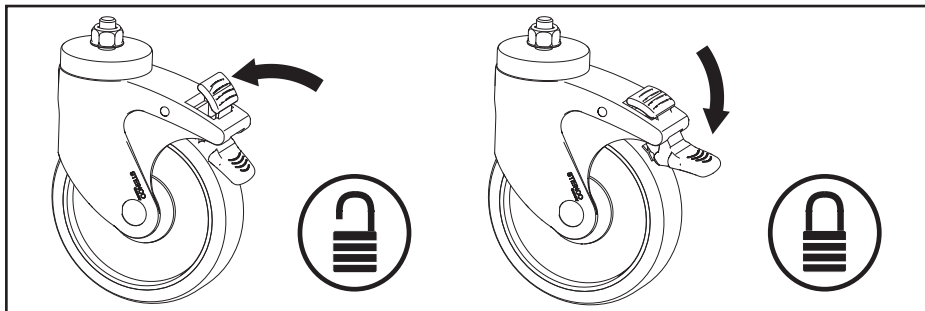
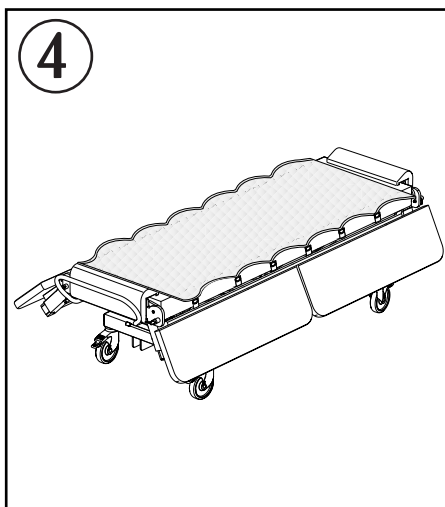
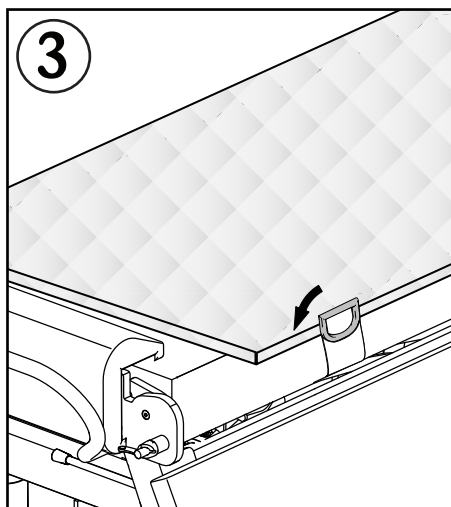
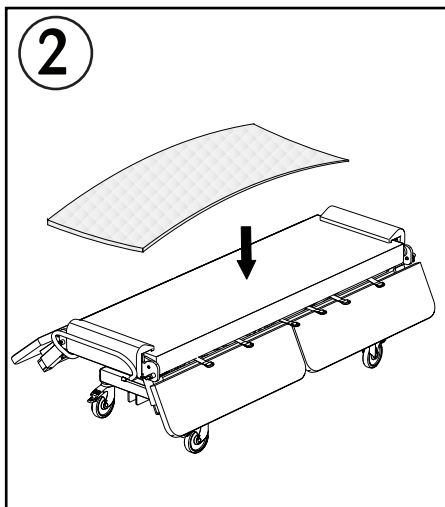
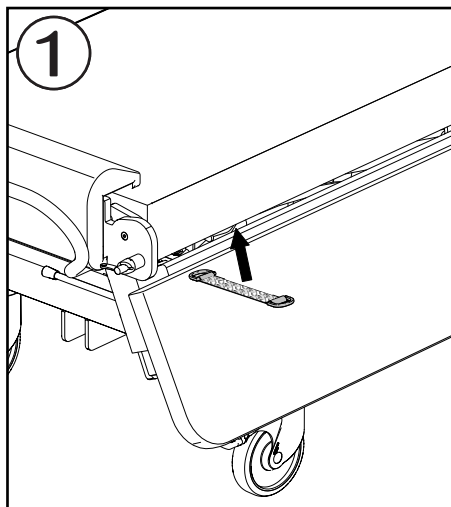
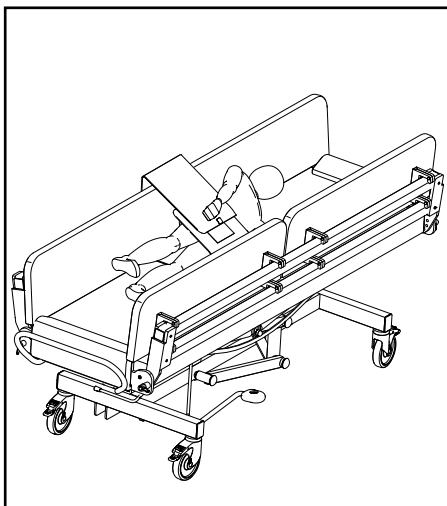
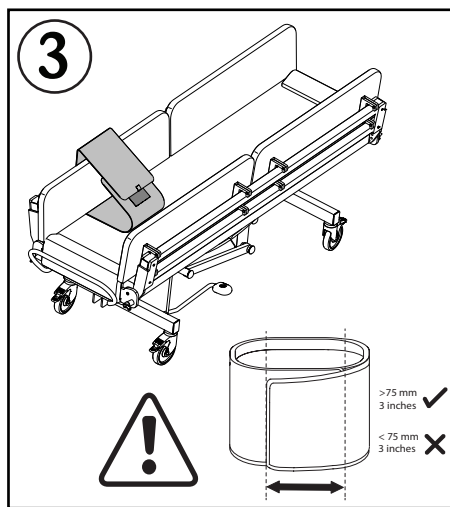
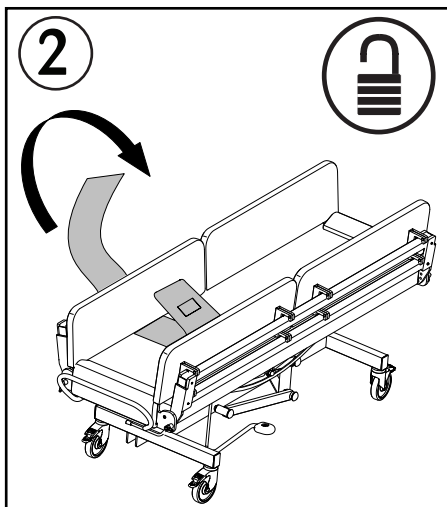
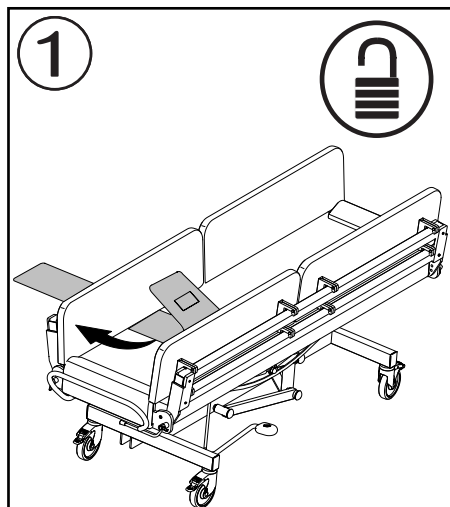
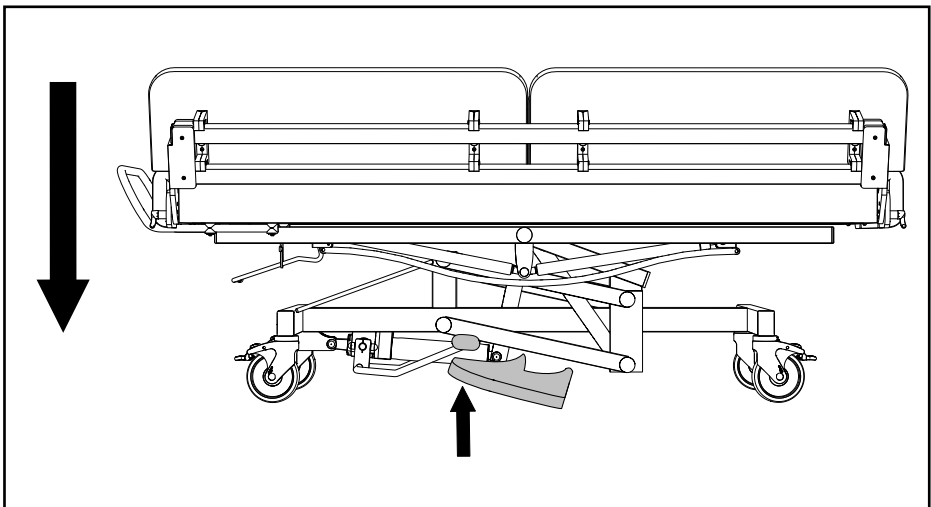
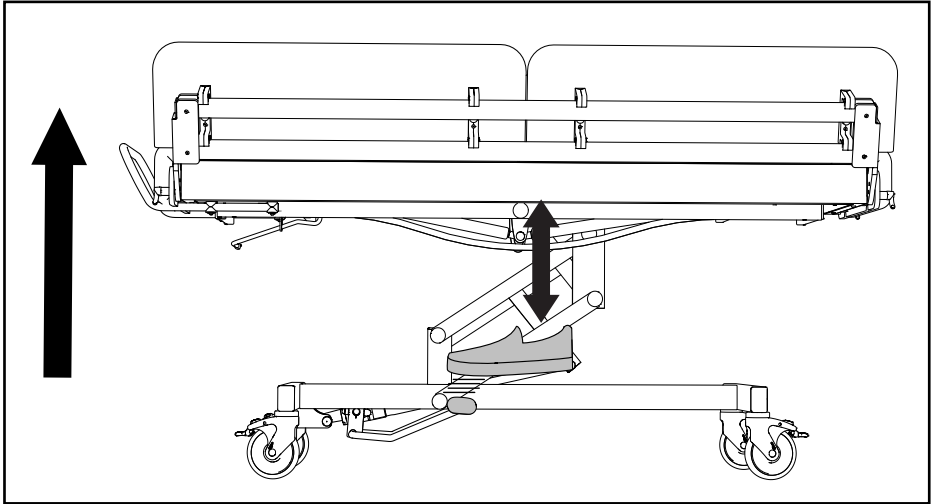
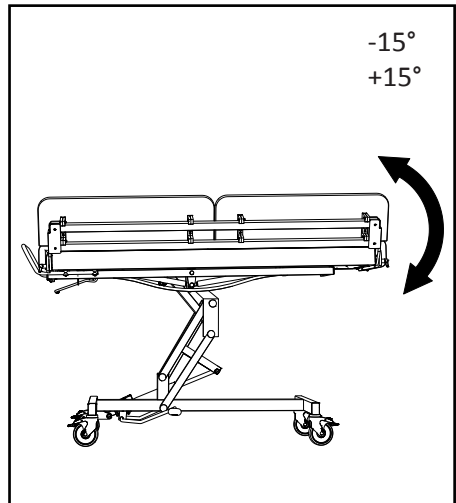
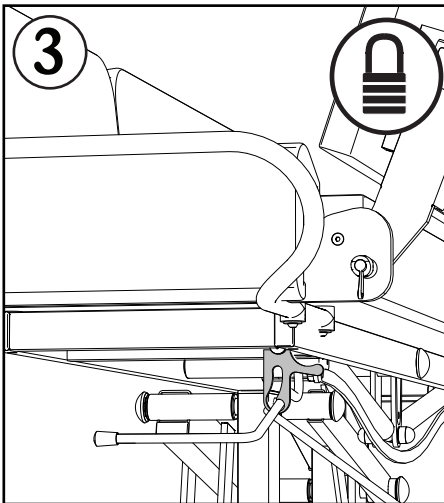
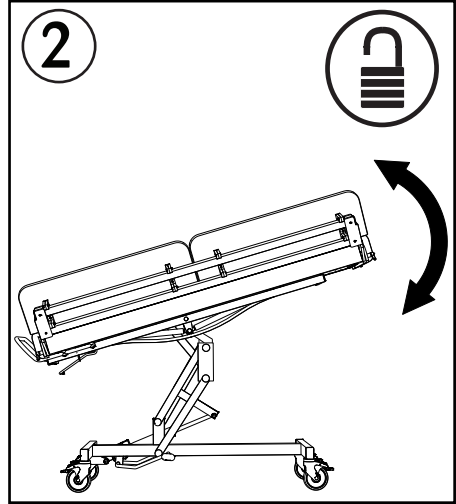
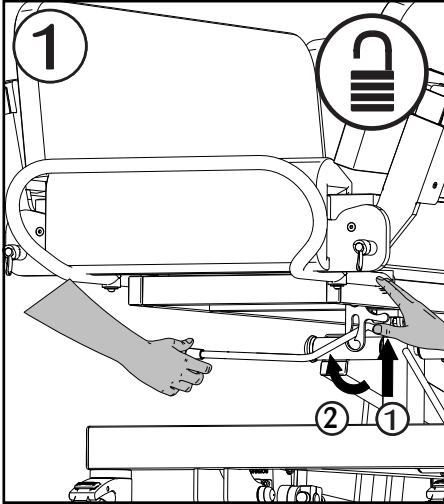
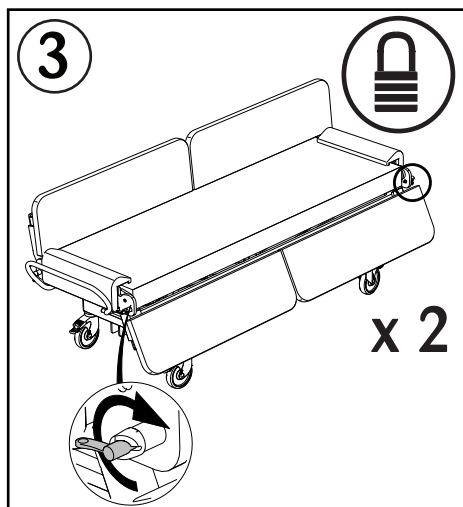
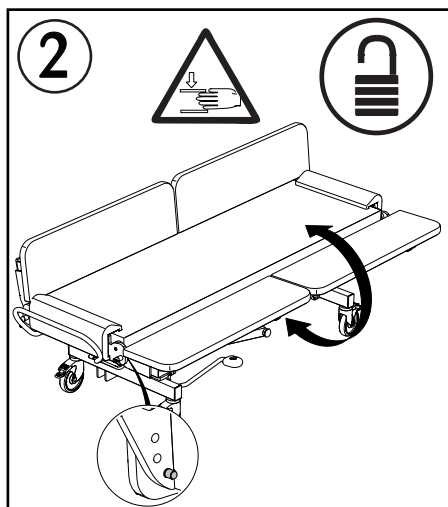
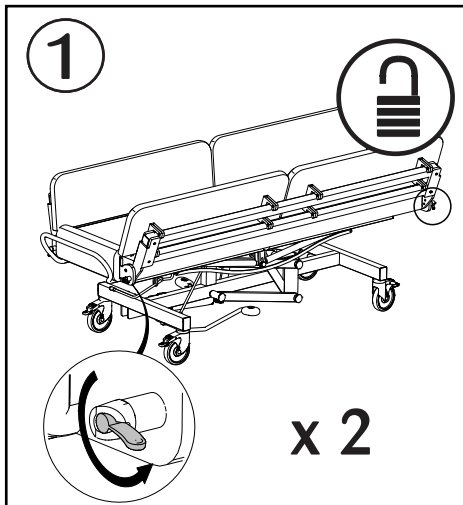
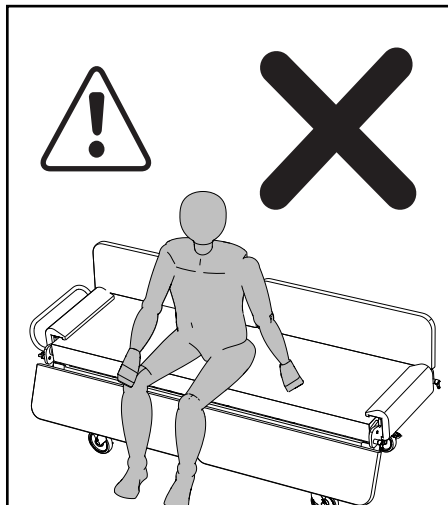
Fault	Possible Cause	Remedy
Couch is unstable	Uneven floor	Move couch to flat floor
Hydraulic pump will not work (502H only)	Pump fault	Replace pump
Hydraulic pump sinks (502H only)	Valve stuck	With couch at max height pump pedal for a further 20-25 strokes to clear valve
Tilt in space will not work (manual versions)	Gas spring faulty	Replace gas spring

Notes

notas
notatki

Jenx Limited, Wardsend Road, Sheffield S6 1RQ, United Kingdom
Tel: +44 (0) 114 285 3376
Email: export@jenx.com
www.jenx.com

Jenx Limited, Wardsend Road, Sheffield S6 1RQ, United Kingdom
Tel: +44 (0) 114 285 3376
Email: export@jenx.com
www.jenx.com

Jenx Limited, Wardsend Road, Sheffield S6 1RQ, United Kingdom
Tel: +44 (0) 114 285 3376
Email: export@jenx.com
www.jenx.com