

Seating

Standing

Sleeping

Therapy

## Standz 2 by Jenx

A versatile standing system offering prone or supine standing that accommodates abducted or neutral leg positioning for children between 5 - 14 years



### Key features

- Tool free adjustments
- Highly configurable
- Individual leg abduction and adjustment
- Supine, prone and upright positioning
- Antimicrobial covers to aid infection control in multi-user environments
- Wide range of growth

Age range: 5 to 14 years  
Max user height prone/supine: 63 inch  
Max user weight: 154 lbs  
Individual leg abduction: Neutral to 30°

## Always Supportive...



# Standz 2 by Jenx

A versatile standing system offering prone or supine standing that accommodates abducted or neutral leg positioning.

More than just a scaled up version of the original Standz, size 2 offers function and accessories that are appropriate for an older, taller, heavier and stronger user whilst keeping the much loved simplicity of adjustment and use found on Standz.

Functional features and accessories recognise the needs of older users and also keep the product simple and safe for care providers to use. A number of accessories and features have been developed as a result of feedback from global clinicians and parents. Continuing with the core principals of clinically correct positioning, Standz 2 allows for clinicians to support older children and adolescents as they move from postural development through to postural maintenance.

## INNOVATIVE FEATURE

### Tool Free Adjustments

No more hunting for the right sized allen key, cleverly designed wing knobs and adjustment pins offer the ability to make adjustments with minimal effort – meaning growth adjustments have never been easier.

## INNOVATIVE FEATURE

### Antimicrobial Covers

As standing aids the digestive system and gravity can take it's course, our covers have been designed to wipe clean, they also contain an antimicrobial agent to aid infection control, ideal for clinic or multi-user environments.



## INNOVATIVE FEATURE

### Cable Operated Tilt-in-Space

Standz 2 offers a dedicated hand-operated lever which allows adjustment of the frame through infinite angle adjustments from 0-90° with the built-in angle gauges. A safety latch on the lever prevents the frame angle being adjusted accidentally.

## INNOVATIVE FEATURE

### Trunk Support Options

Optional **Shoulder Protractors** encourage correct shoulder and arm positioning to help promote a functional hand position for play or learning. An optional **Waistcoat Harness** provides additional trunk support for children with low tone.

## INNOVATIVE FEATURE

### Frame Mounted Carer Foot Inlays

To enable the carer a stable point of contact when adjusting the tilt-in-space angle, the frame features built-in foot inlays.

## INNOVATIVE FEATURE

### Angle Gauges

Built in angle gauges for frame angle and individual leg abduction - easy to read gauges enable precise positioning and allow for the correct position to be repeatedly achieved.

## INNOVATIVE FEATURE

### Individual Leg Adjustment

Adjustment for both knee and footplate height – ideal for accommodating leg length discrepancy. This tool free adjustment can be made in seconds and securely clamps into place using our clever clamping marquee design.

## INNOVATIVE FEATURE

### Individual Leg Abduction

Standz 2 offers the ability to independently adjust either leg from 0° – 30° of abduction depending on the child's clinical needs. Simply adjusted via a locking pin, this action is quick and simple to adjust for each standing session. An incorporated angle gauge makes it easy to see the degree of abduction for each leg which can be adjusted in 5° increments.

## Standz 2



### As Standard Standz 2 Frame comes with:

- 4 lockable swivel casters
- Tilt mechanism with dedicated release lever on handle, incorporated safety lock
- 2 leg posts clearly marked with numbers for easy positioning of leg supports
- 0°– 30° independent abduction movement of leg posts in 5 degree increments with easy to use locking plunger
- Individual footplates with plantar and dorsiflexion movement
- Angle gauge to read frame angle – frame angle range of 5° – 90°
- Pelvic support board and cover pad in a choice of 3 colours
- Trunk support board and cover pad in choice of 3 colours

In addition to the base you will need to add the following accessories to ensure a minimum safe configuration.

**Prone** – sandals, knee support, pelvic support & trunk support.

**Supine** – sandals, knee support, knee support straps, pelvic support, trunk support & head support.

## Always Configurable Accessories



### Flat Pad Head Support

A simple yet effective head support pad for use in supine for children with reasonable head control. It can also be used in prone to extend the height of the chest pad.

MH09-XX



### Multigrip Head Support (supine standing only)

The original and innovative Jenx Multigrip™ head support is available in 2 sizes for all Jenx seating/standing systems.

2 Link MH0101  
3 Link MH05



### Molded PU Head Support

Simple headrest for children who require minimal head support. Comes mounted on a flexible bracket.


MH07-XX



### Pelvic Support Board

For use in prone positioning only, the pelvic support board offers a full contact surface to facilitate the full extension of the pelvis and prevent pelvic rotation, pelvic obliquity and unwanted hip flexion.

STZ 032-XX

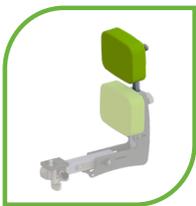

### Pelvic Support Board - Upper Back Pad

Used in conjunction with the pelvic support board, the back pad offers support for the child's upper back. A head support can also be mounted to the back pad to help keep a neutral head position for those children who hyper extend their neck when standing in prone.


STZ 016-XX



### Shoulder Support with Protractors

Shoulder protractors mounted to a flat shoulder pad, offers angle adjustment and promotes shoulder girdle stability. Required to achieve maximum user height.


STZ 034-XX

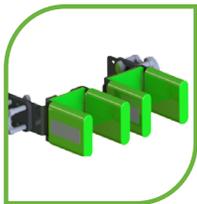

### Standard Knee Support

Individual knee supports offering the ability to accommodate knee flexion when used in supine (requires knee straps when used in supine). Width adjustable to correctly gutter the knee in position.


STZ 007-XX

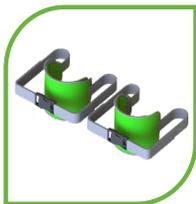

### Standard Knee Strap

Intended for use in conjunction with the standard knee supports, the standard knee strap simply holds the knee in the desired position and prevents knee flexion from occurring.

MU54-XX Size 1  
MU68-XX Size 2

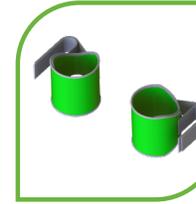

### Rotation Control Knee Straps

Intended for use in conjunction with the standard knee supports, the rotation control straps allow for a cuffing of the child's leg and allow for control of internal or external knee rotation.

MU45-XX Size 1  
MU69-XX Size 2



### Flip Away Flexi Laterals with strap

Flexible lateral supports with incorporated support strap. Wrap around the body to give firm yet comfortable positioning. Can also be used at the pelvis if desired.

STZ 025-XX



### Lateral Supports with Strap

Firm lateral support pads offer stability to the trunk, pelvis or both together, and the incorporated support strap can be used to facilitate extension or rotation. Flip away to aid transfers.

STZ 024-XX



### Multi-Axis Knee Supports

Use for supine standing only. For use where there is the need to accommodate fixed knee flexion.

STZ 017-XX Size 1  
STZ 023-XX Size 2



### Multi-position Tray

Cleverly designed to be used in both prone and supine, the tray is angle adjustable to ensure the correct position in either application.

STZ 1028-XX



### Tray Extension Wings

Strongly recommended for use in supine and just as effective when used in prone, these wings attach to the rear of the tray to offer a support surface for the child's elbows and prevents arms from slipping off the tray

STZ 035



### Sandals (Pair)

Available for Standz in either size 1 or 2, these sandals ensure solid foot placement to give the foundation for effective leg positioning and safe standing.

Size 1 S16 6¾"(d) x 3¼"(w)  
Size 2 S2 8½"(d) x 3¾"(w)



### Tray Elbow Blocks

Aid in preventing retraction of the child's arms and promote a functional arm and hand position – used in conjunction with Tray Extension Wings.

**UBEB**



### Waistcoat Harness

Gives full trunk and shoulder girdle stability when using the Standz 2 in the supine position. Available in 2 sizes.

**HB62** Size 2  
**HB64** Size 3



## Standz 1 by Jenx Technical Specification

	Units	Size 2 Prone STZ-BS1	Size 2 Supine MUS01-BS1
Age Range (approx)		5 yrs - 14 yrs	5 yrs - 14 yrs
User Height	inch	39 – 63	39 – 63
User Weight (max)	lbs	154	154
Chest Width	inch	8.7 – 14.2	8.7 – 14.2
Hip Width	inch	9.8 – 14.6	9.8 – 14.6
Height (sandal plate to top of upper support board)	inch	32.3 – 45.7	32.3 – 45.7
Height (floor to top of upper support pad, in upright)	inch	49	49
Height (floor to support board, at horizontal)	inch	N/A	35.4
Kneeblock Width	inch	8.7	8.7
Footplate Angle (dorsi-plantar)	degrees	15° – 15°	15° – 15°
Leg Abduction (each)	degrees	0° – 30°	0° – 30°
Tilt-In-Space Angle	degrees	0° – 90°	0° – 90°
Base Footprint (width x length)	inch	29.9 x 42.1	29.9 x 42.1
Overall Weight (approx. range with accessories)	lbs	90 -110	90 -110

## Standz Color Range



#### EMAIL & WEB:

sales@adaptiveimports.com  
www.adaptiveimports.com

#### PHONE & FAX:

Tel: (877) 767-9462  
Fax: (877) 394-3465

#### ADDRESS:

2760 Circleport Dr.  
Erlanger, KY 41018

distributors of

