

HEADALOFT™

HEADALOFT Product Description

HEADALOFT is a patented, innovative solution for adults and children who lack head control. Made of high-quality, durable fabric, **HEADALOFT** comfortably and gently holds the head upright in a versatile, non-restrictive manner. Suitable for all seating and mobility devices equipped with a head rest, **HEADALOFT** is the ultimate head support add-on solution. It is easy to install, can be transferred quickly between wheelchair and car seat or any other type of seat that has a headrest to which the **HEADALOFT** can be attached. **PDAC Code Verified: HCPCS K0108**

HEADALOFT Benefits:

- Improves body symmetry
- Increases ease of breathing
- Promotes easier feeding and swallowing
- Enhances communication and social engagement
- Enables eye gaze control and increased interaction with others
- Provides ergonomic neck support
- Allows for quality napping while in the wheelchair, stroller or car seat
- Prevents neck pressure sores



Medical experts, caregivers, therapists, and parents agree that **HEADALOFT** has significantly improved not only the health and well-being of their patients or children, but has also resulted in a higher quality of life, increased social interaction, and more independence.

Technical Specifications

HEADALOFT is made using the highest quality materials. It comes with three sized (S, M, and L) forehead straps to provide anterior support. Soft and comfortable, the product is machine washable and standards-compliant.

box size: 51*15*5
6 units per carton



HEADALOFT Occipital Support

The innovative occipital support cushion is an essential add-on that offers additional support and comfort.

It is easily and quickly installed and is machine washable.



Developed by:

Headovations
HEAD SUPPORT SOLUTIONS

In Partnership with:

ALYN HOSPITAL
Pediatric & Adolescent
Rehabilitation Center



HEADALOFT™

Installation and Maintenance Guidelines

Installation Instructions:

Installing **HEADALOFT™** on any seating and mobility device equipped with headrest is very easy and can be accomplished very quickly. Follow these simple instructions step-by-step to install **HEADALOFT™**.



An occipital support cushion is added to provide additional support.

Maintenance:

HEADALOFT™ requires no real maintenance. It is extremely durable and requires only occasional washing. It can be machine washed in temperatures up to 30°C / 86°F, however you should not use bleach as this can damage the appearance and material.

After washing **HEADALOFT™**, you should not iron it or put it in the dryer. Instead, allow it to dry naturally. It should also not be dry cleaned.

Storage:

When not in use, **HEADALOFT™** should be stored in a clean, dry area.

distributors of



EMAIL & WEB:
sales@adaptiveimports.com
www.adaptiveimports.com

PHONE & FAX:
Tel: (877) 767-9462
Fax: (877) 394-3465

ADDRESS:
2760 Circleport Dr.
Erlanger, KY 41018

HEADALOFT™